

Moveo S.r.l.

Via Monsignor G. Fortin, 38
35128, Padova, Italia
+39 049 26144 27
VAT Number: IT 05236760285
info@moveowalks.com
moveowalks.com



ExoBand Data Sheet

DESCRIPTION OF THE MEDICAL DEVICE

WARNINGS

The medical device must be prescribed and used under medical supervision. The orthopaedic specialist is the expert of reference both for the application and for information relating to safe use. To guarantee the efficacy, tolerability and proper operation of the product, it must be applied and adjusted with the utmost accuracy. Any modification of the structure of the device or adjustment of the latter must be prescribed by a doctor and implemented by an orthopaedic specialist. In hypersensitive individuals, direct contact with the skin may cause redness or irritation. If you experience pain, swelling, or any other abnormal reaction, contact your doctor immediately.



The medical device is designed exclusively to support human walking. Any unintended use is not recommended (such as using it when travelling in a car, riding a motorbike or bicycle, or even in the water, etc.). Any damage resulting from improper use is the responsibility of the customer.

The device consists of a belt, two leg loops and two tensioners which together with a toothed guide join all these elements. The device is available in 4 interchangeable sizes (XS, S, M and L).

Class I Medical device.

INTENDED USE AND UNINTENDED USE OF THE DEVICE

Sphere of application and intended use.

The medical device must be used exclusively as:

- A walking aid for the elderly and people with motor impairments caused by disabling diseases and/or health conditions.
- An aid to improve improper or pathological posture.

Unintended use

The medical device is not designed for any use other than those described in the paragraph "SPHERE OF APPLICATION AND INTENDED USE". Furthermore, the following uses are absolutely forbidden:

- Use of the medical device by individuals who have not read the manual.
- A medical prescription is recommended for use by individuals suffering from serious diseases such as musculoskeletal system tumours, neuromuscular pathologies, spinal pathologies, hernias, etc.
- Use by pregnant women
- Use in direct contact with the skin

Limitations of the medical device

- The medical device is not able to cure serious spinal deformations (dysmorphisms) or other dysmorphisms (it can only reduce the outcomes).
- The medical device can be used for long periods of time, that is, until it shows signs of wear or damage that are such as to affect its structure, safety, and operation. In this case and when in doubt, consult the seller immediately.
- The medical device can reach its maximum efficiency after a gradual period of use according to the psychophysical conditions of the user.

Washing



Machine wash in cold water on a delicate cycle after removing the elastic components from the leg loops as described above. Leave to dry on a drying rack, away from heat sources.

Do not use dryer. Do not use fabric softener. Do not dry clean. Do not iron.

BASIC UDI-DI: 805934045EXOBAND0003N

EMDN CODE: Y061215

EMDN DESCRIPTION: HIP ORTHOSES (INCLUDING ABDUCTION ORTHOSES)

GMDN CODE: 36228 (HIP ORTHOSIS)