

Moveo S.r.l.

Via Monsignor G. Fortin, 38
35128, Padova, Italia
+39 049 26144 27
P.I. VAT IT 05236760285
info@moveowalks.com
moveowalks.com



Technical Data Sheet ExoBand

MEDICAL DEVICE DESCRIPTION

To ensure effectiveness, tolerability, and proper functioning, the device must be applied with the utmost care. Any structural modification to the device should only be recommended by qualified personnel. In case of pain, swelling, inflammation, or any other abnormal reaction, consult your doctor immediately.



The medical device is designed solely to support human walking. Any use in other circumstances is strongly discouraged (e.g., while driving a car, riding a motorcycle or bicycle, swimming, etc.). Any damage resulting from improper use is the sole responsibility of the customer. The device consists of a belt, two thigh bands, and two tensioners that, when paired with the toothed guide, connect these parts. The device is produced in five interchangeable sizes (XS, S, M, L, XL).

Class I Medical Device.

INTENDED USE AND NON-INTENDED USE OF THE DEVICE

Scope and Intended Use

- Hip orthosis for supporting and improving walking in individuals with motor disabilities caused by debilitating diseases and health conditions, including neurological disorders (e.g., Parkinson's disease, multiple sclerosis, neuromuscular diseases, cerebrovascular diseases, outcomes of central nervous system trauma, cerebral palsy) or orthopedic conditions.
- Hip orthosis to facilitate walking in individuals with motor slowing caused by disabling diseases and/or health conditions.
- Hip orthosis for improving incorrect or pathological posture.

Non-Intended Use

No use other than those described in the "SCOPE AND INTENDED USE" section of the manual is permitted. For use by individuals with serious conditions, such as locomotor system tumors, neuromuscular disorders, spinal conditions, hernias, etc., a medical evaluation is recommended.

It is also strictly prohibited to:

- Use by individuals who have not read the manual
- Use by pregnant women

Limitations of the Medical Device

- The medical device is not capable of correcting severe spinal deformities (dysmorphisms) or other dysmorphisms; it can only reduce their effects.
- The medical device can be used for an extended period, as long as it shows no signs of tears or damage that could compromise its structure, safety, or functionality. In such cases, or if there are any doubts, it is recommended to consult the seller immediately.
- The medical device may reach its maximum efficiency after a gradual period of use, depending on the user's physical and psychological condition.

Cleaning



. Hand washing, machine washing, or sanitizing with a disinfectant spray for technical fabrics is permitted, as outlined in the "Cleaning" section of the User Manual. Do not use bleach or other harsh chemicals that may damage the device.

- Spin drying is prohibited.
- Regularly check the device's condition to ensure there are no damages or signs of wear that could compromise its functionality.
- Do not iron.
- Always follow the manufacturer's specific instructions for maintenance and care of the device.

Basic UDI-DI: 805934045EXOBAND0003N

EMDN Code: Y061215

EMDN Description: Hip Orthosis (including hip abduction orthosis)

REF Code: EXOBAND

GMDN Code: 36228 (Hip Orthosis)